

GET BACK MORE OF WHAT YOU'VE BEEN MISSING. EXPERIENCE HIGH-QUALITY VISION DAY AND NIGHT.²

The **TECNIS Eyhance™** IOL defies conventional thinking about monofocal IOLs by delivering not only high-quality distance vision, but also better low-light image contrast.^{2*}

You live for your favorite activities. Find it hard to focus on signs while driving? Does the TV look blurry? Worried about walking on uneven surfaces?



ASTIGMATISM



Distorts and blurs vision



Can lead to eye discomfort and headaches



Can be addressed with a toric IOL




Other solutions include prescription glasses and contacts made for astigmatism or an additional surgery


TECNIS
Eyhance™ Toric II IOL
with TECNIS SIMPLICITY® Delivery System

Toric II

TECNIS Eyhance™ Toric II IOLs address astigmatism as part of cataract surgery.³

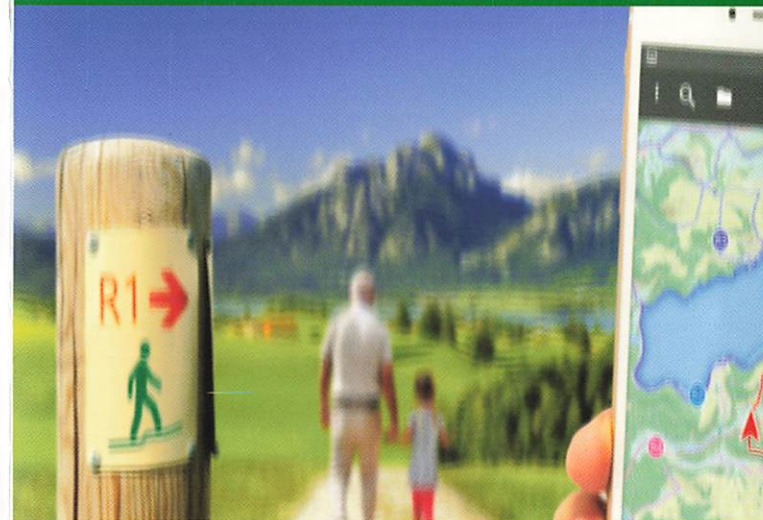

With presbyopia and astigmatism

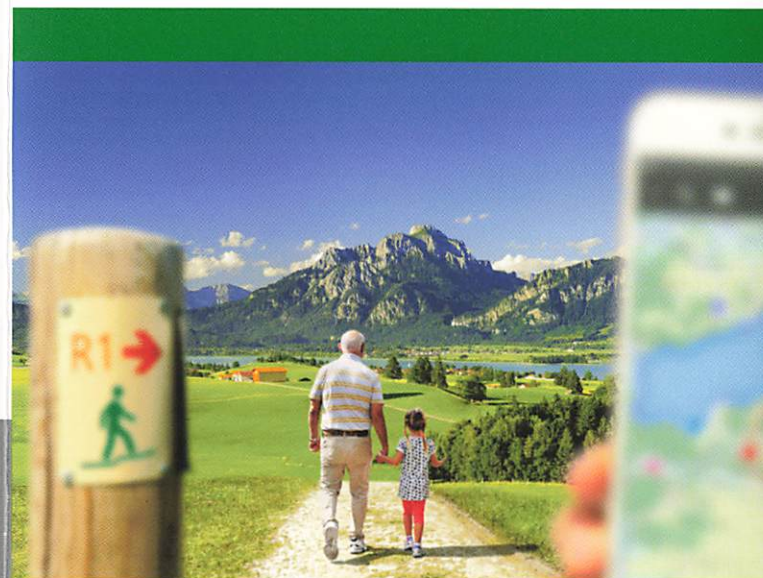

With **TECNIS Eyhance™** Toric II IOLs

These images are for illustrative purposes only and do not represent actual data derived from studies. These illustrative simulations are intended to help you better understand your vision in certain eye conditions.

BENEFITS OF TECNIS EYH



Helps you see objects at distance like traffic light performances and speed



May increase safety in situations such as driving



Designed for high-quality vision. You may still need glasses closer.

Talk to your doctor about the life you want to live. Find out if TECNIS Eyhance™ Toric II IOLs are right for you.

^{*}compared to Alcon Acrysof IQ IOLs¹



TECNIS Eyhance™ Toric II IOL

with TECNIS SIMPLICITY® Delivery System

Toric II

ABOUT TECNIS EYHANCE™ TORIC II IOL (Continued)

INDICATIONS and IMPORTANT SAFETY INFORMATION for TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOLs with TECNIS Simplicity® Delivery System
Rx Only

INDICATIONS FOR USE: The TECNIS Simplicity® Delivery System is used to fold and assist in inserting the TECNIS Eyhance™ IOL for the visual correction of aphakia in adult patients in whom a cataractous lens has been removed by extracapsular cataract extraction. The lens is intended to be placed in the capsular bag. The TECNIS Simplicity® Delivery System is used to fold and assist in inserting the TECNIS Eyhance™ Toric II IOLs for the visual correction of aphakia and pre-existing corneal astigmatism of one diopter or greater in adult patients with or without presbyopia in whom a cataractous lens has been removed by phacoemulsification and who desire reduction in residual refractive cylinder. The lens is intended to be placed in the capsular bag.

CONTRAINDICATIONS: There are no known conditions under which the TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOLs should not be used.

RISKS: There are risks to routine cataract surgery. This is irrelevant to the lens you choose. The problems could be minor, temporary, or affect your vision permanently. Complications are rare. These may include worsening of your vision, bleeding, or infection. Similar to other monofocal IOLs, patients implanted with the TECNIS Eyhance™ or TECNIS Eyhance™ Toric II IOLs may require glasses for some tasks. Discuss all risks and benefits with your eye doctor prior to surgery.

WARNINGS: Pre-existing diseases or conditions may place you at higher risk of experiencing complications (e.g., more difficult recovery) after routine cataract surgery. Examples of pre-existing diseases or conditions include, but are not limited to: ocular inflammation, surgical difficulties at the time of cataract extraction, suspected eye infection, congenital bilateral cataract, history or predisposition to retinal detachment, patients with only one good eye with potentially good vision, medically uncontrollable glaucoma, diabetes, heart disease, and previous trauma to your eye good eye with potentially good vision, medically uncontrollable glaucoma, diabetes, heart disease, and previous trauma to your eye. The TECNIS Eyhance™ Toric II IOL corrects astigmatism when it is placed correctly in the eye. There is a chance that the TECNIS Eyhance™ Toric II IOL could be placed incorrectly or could move within the eye, resulting in visual distortions. A second surgery may be needed to properly position the lens. These lenses have not been evaluated for use in children.

PRECAUTIONS: The effects of the TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOL optical design on quality of vision, contrast sensitivity, and subjective visual disturbances (glare, halo, etc.) have not been evaluated clinically. If your eye is not healthy (including glaucoma), your vision may not be good even after your cataract is removed. In this case, you may not get the full benefit of the TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOLs. Before surgery, your eye doctor will check if you have any eye diseases that may influence your IOL selection. In rare instances, the TECNIS Eyhance™ and TECNIS Eyhance™ Toric II IOLs may make some types of retinal treatment (e.g., retinal tear repair) more difficult. Take all prescribed medicines and apply eye drops as instructed to avoid inflammation and infection. Your eye doctor may tell you if you should avoid certain activities after surgery.

SIDE EFFECTS: Side effects of cataract surgery with the TECNIS Family of 1-Piece IOLs may include swelling/thickening of an area of the retina, and/or the need for a second surgery to reposition the lens, repair the retina, or remove the new lens.


ATTENTION: Reference the Directions for Use for a complete listing of Indications and Important Safety Information.

1. What is astigmatism? Your Sight Matters Web site. <http://yoursightmatters.com/what-is-astigmatism/>. Accessed December 7, 2017 REF2017OTH0308.
2. Data on File, Johnson & Johnson Surgical Vision, Inc. Jan. 2021. DOF2021CT4006.
3. TECNIS Eyhance® Toric II with TECNIS Simplicity™ Delivery System, Model DIU - US DFU - Z311525E Rev. A, Jan. 2021. REF2021CT4007
4. Data on File, Johnson & Johnson Surgical Vision, Inc. 2021 DOF2021CT4002

© Johnson & Johnson Surgical Vision, Inc. 2021
PP2021CT4011 v2.0

Johnson & Johnson VISION

RESTORE WHAT YOU'VE BEEN MISSING.

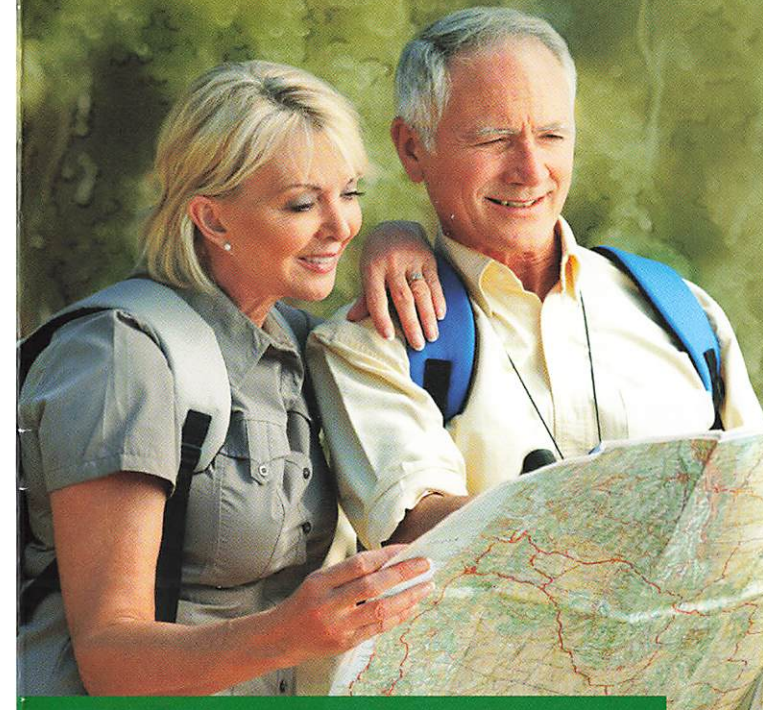

The revolutionary IOL for Astigmatism

**LIVE THE LIFE
YOU WANT TO SEE.**

**TECNIS
Eyhance™ Toric II IOL**
with TECNIS SIMPLICITY® Delivery System

Toric II

About TECNIS Eyhance™ Toric II IOL: Physicians considering use of the TECNIS Eyhance™ Toric II IOLs should refer to the Directions for Use labeling for a complete list of indications and safety information.


See reverse for continued Indications and Important Safety Information.

Johnson & Johnson VISION



Astigmatism is a lengthy word with a simple explanation: the shape of your eye is more oblong than circular, which prevents you from focusing clearly.

Most people are born with some degree of astigmatism,¹ but it can affect some more than others.

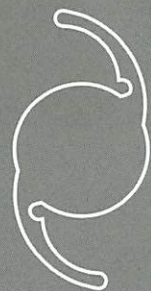

If you have both astigmatism and cataracts, you may have the opportunity to address them simultaneously with a toric IOL.